

# DENTESHIELD™ SURFACE SEALANT APPLICATION INSTRUCTIONS

STEP 1  
STEP 2  
STEP 3  
STEP 4  
STEP 5

CLEAN  
ETCH  
RINSE  
APPLY  
CURE





## PROPHY THE ENAMEL SURFACES

Use only oil-free prophy pastes



## ETCH USING ETCHING AGENT


Dispense or apply etching agent of choice following related instructions.

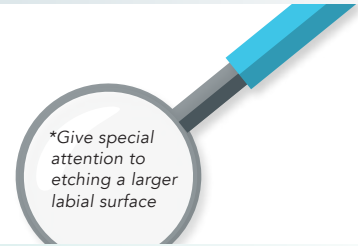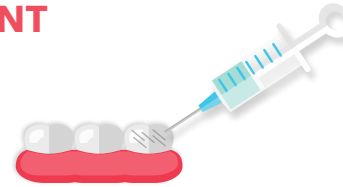

## RINSE THOROUGHLY WITH WATER

Rinse each tooth with copious amounts of water for 10 seconds to stop the etching process (20 seconds with gel etch)

10 SECONDS

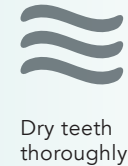

## APPLY SURFACE SEALANT

Dispense a large drop or two of DenteShield™ sealant into a mixing well



With a brush, apply a thin uniform layer of sealant to the exposed labial tooth surface.

**NOTE** If preferred, use air gun to evenly spread sealant

## CURE SURFACE SEALANT

Light cure each tooth for approximately 20 seconds.

20 SECONDS



**NOTE** Curing times depend on the intensity of the curing light. The total curing time can range from 6 seconds to 20 seconds. Refer to the light manufacturer for recommended curing times.