

DENTESHIELD™ Pit & Fissure Sealant

INSTRUCTIONS FOR USE

PREPARATION

- Clean surface to be treated. Perform prophylaxis on the occlusal surface. A non-fluoride, oil-free prophy paste or a mixture of pumice and water should be used.
- Rinse well with water. Isolate the area with cotton rolls. Dry tooth with oil-free/moisture-free air for 15 seconds.
- Apply etchant. Etch area at least one minute. Rinse etched area for 15 seconds and dry thoroughly. If area does not appear chalky and frosty, etch again. Or Etch area in accordance with instructions for etch product of choice.

APPLICATION

- Apply Light Cure Pit & Fissure Sealant to the occlusal surface. Avoid air bubble entrapment. Two drops of sealant material is sufficient to treat the occlusal surface on one quadrant.
- Using a dental light curing unit, expose all coated surfaces for 45 seconds, keeping the light 1-2 mm from the surface.
- After sealant has set, wipe surface with cotton roll. Check for hardness and retention.

PRECAUTIONS & WARNINGS

- Etchant contains 37% phosphoric acid. Avoid contact with skin, soft tissue, and eyes. Flush with water in case of contact with skin or eyes. Seek medical attention in case of eye contact.

STORAGE: Store at temperature of 38 - 76°F. Place at room temperature 5-10 minutes before usage.

FOR PROFESSIONAL USE ONLY



SelenBio Dental, Inc
4009 Banister Lane, Suite 370
Austin, TX 78704

833-4SELDOX
selenbio.com
MADE IN U.S.A.



SelenBio Dental, Inc
4009 Banister Lane, Suite 370
Austin, TX 78704

MANUFACTURED FOR SELENBIO DENTAL, INC.